

How to setup the SmartNode?

What do the icons mean?

How to paste the ThermoSensor?

Download the latest User Manual from:

www.cadisense.com

Support Hotline:

+65 6276 2676



CE 0434



Medical Device Safety Service GmbH
Schiffgraben 41 30176 Hannover Germany

Complies with
IDA Standards
DB 102490



CADI SCIENTIFIC PTE LTD, 31 Ubi Road 1, #07-01A, Singapore 408694,
AZTECH TECHNOLOGIES PTE LTD, 31 Ubi Road 1, Singapore 408694,
SHANGHAI HUYOU MEDICAL ELECTRODE LTD, 45 Lane 1398 Zhenchen Rd Baoshan Shanghai, China

v3.21.3

Cadi.Sense

Quick Start Guide

How to setup the SmartNode?

Before you start...

Please write down the info shown on your SmartNode plug here:

SSID (e.g., CADI_XXXX): _____

PWD (8 chars): _____

SmartNode ID: _____

Step 1

Plug in SmartNode in the room where the patient will be resting. Then switch it on.

(Note: SmartNode takes about 30 secs to start up.)

Step 2

Turn on WiFi on your mobile device. Then connect it to SmartNode using the above SSID and password (PWD, 8 chars).

Step 3

Go to web browser and enter:

http://10.10.10.10

You'll see this →

Quick setup is done!



If you'd like to **access Cadi.Sense outside your home**, please continue:


Step 4

Click the  icon, then click "Connect to WiFi Router".

You'll see this →

Click on your router's SSID, key in your router's password, then click "Apply".

(Please be patient as it will take 2 mins.)

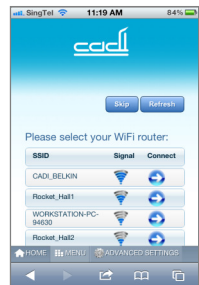

Step 5

Go to **www.cadisense.com**, then click "Access Cadi.Sense". Key in your SmartNode ID (see before Step 1), password is "**cadi!**", default port is "**50000**", then click "Login".


Cadi.Sense

Quick Start Guide


What do the icons mean?



Main Page
Sensor Selection

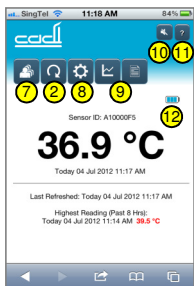

Home Page
Temperature Monitoring

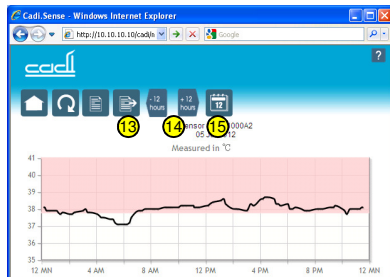

Temperature Chart

1. Go to Home Page (Temperature Monitoring)
2. Refresh
3. Advanced Settings
4. Set Timezone, Date & Time
5. Add / Remove Sensor
6. Click to go Home Page
7. Go to Main Page (Sensor Selection)
8. Sensor Settings (enter name, choose °C / F, set temperature threshold, alert message)
9. Temperature Chart / History (note: chart will appear only after 15 mins; "Lo" means below 32.1 °C; "Hi" means above 42.0 °C)
10. Toggle Sound Alert (only for iPhone / iPad)
11. Help
12. Sensor Battery Status
13. Export Temperature History Data (only for laptop)
14. Shift Forward / Backward 12 Hours
15. Jump To Date

Cadi.Sense

Quick Start Guide


How to paste the ThermoSensor?

Step 1

Apply a small amount of body lotion onto the lower abdomen area.

Step 2

Take a Stick On and peel off the round paper liner to expose the adhesive.

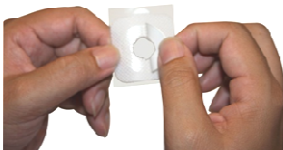

Step 3

Paste the ThermoSensor on the adhesive, with the metal part facing outward and away from the adhesive.



Step 4

Remove the rectangular backing liner.



Step 5

Paste the sensor by sticking the adhesive part to the inside of the diaper/undergarment, so that the sensor's metal part will touch the lower abdomen (left or right) when the diaper/ undergarment is properly worn.



(Note: ThermoSensor takes 10 mins to warm up, temperature is wirelessly sent every 30 secs.)

Keeping the sensor

Please remember to keep your sensor back into the storage case properly to switch off the sensor. This will save battery power and make your sensor last longer!